



Upgrade · The Home Creator Way

Bed Handle

MODEL: HFK-5124-1 , HFK-5115-1 , HFK-5116-2

**VEVOR Support
Center**



HFK-5124-1



HFK-5115-1



HFK-5116-2



HFK-5124-1



HFK-5115-1



HFK-5116-2

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.



Warning-To reduce the risk of injury, user must read instructions manual carefully.

SAFETY INSTRUCTIONS

WARNING:

Read this material before using this product. Failure to do so can result in serious injury.

Use precautions

TO PREVENT SERIOUS INJURY AND DEATH FROM TIPPING:

1. DO NOT SIT OR STAND ON THIS ITEM.
2. This product is not a toy. Do not allow children to play with or near this item.
Please keep small spare parts out of children's reach.
3. Use as intended only.
4. Inspect before every use; do not use if parts are loose or damaged.
5. Keep away from sharp points, blades and other items, so as not to scratch the product.
6. The product complies with relevant standards and regulations. Installation of this device must comply with all applicable regulations and it should only be operated in well-ventilated areas. Please read the user manual before installing and using this equipment.
7. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
8. This appliance is intended to be used in household and similar applications such as:
 - staff kitchen areas in shops, offices and other working environments;
 - farm houses;
 - by clients in hotels, motels and other residential type environments;
 - bed and breakfast type environments.
9. Please properly keep product. Check whether the product is abnormal before use. If it is abnormal please stop using it.
10. Please use the product according to the intended use, warn of potential injury from misuse.

Technical Parameters

Model	bracket
HFK-5124-1	5
HFK-5115-1	4
HFK-5116-2	3
FDA Registration No.: 3012579728	

Instructions for Use

APPLICATION OF BED SAFETY RAIL

This Bed Safety Rail is an assistive balance device. It is designed to assist people who are mobile, but require some assistance to get in and get out of bed, or reposition in bed safely. The bed rail is NOT intended for people with very poor mobility, or who are permanently bedridden. It is NOT designed to prevent people from falling out of bed.



CAUTIONS and WARNINGS

1. Please make sure that you read and follow these instructions carefully. Failure to comply may result in injury.
2. Ensure all parts are assembled securely and all spring buttons are properly engaged in position prior to use.
3. The area between an assistive bed rail and mattress creates a potential for serious, life-threatening entrapment. It is possible for a person's head, neck, arm (s), leg (s) or body to become wedged between the side of the mattress and any of the bed rail.
4. The mattress must be tight against the assistive handrail, with no gaps at all times. This should be checked frequently-at least daily.
5. The mattress must be tight against the assistive handrail, with no gaps at all times. This should be checked frequently-at least daily.
6. NEVER use unless the product is tight against the mattress without gaps, and at least 12.5 in. from headboard and foot board.

7. NEVER use with children.

8. NEVER use on toddler, bunk, water, or inflatable beds, beds with soft compressible pads, articulated beds, or on beds with a mattress height taller than 13in (33cm) or shorter than 7 in (17.8cm). ONLY use with twins, queen and king size beds.


9. The product is an auxiliary product. Patients without autonomous ability must be under supervision.




HFK-5115-1



HFK-5124-1

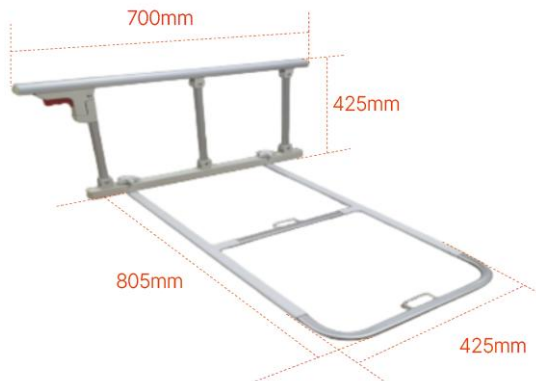

HFK-5116-2

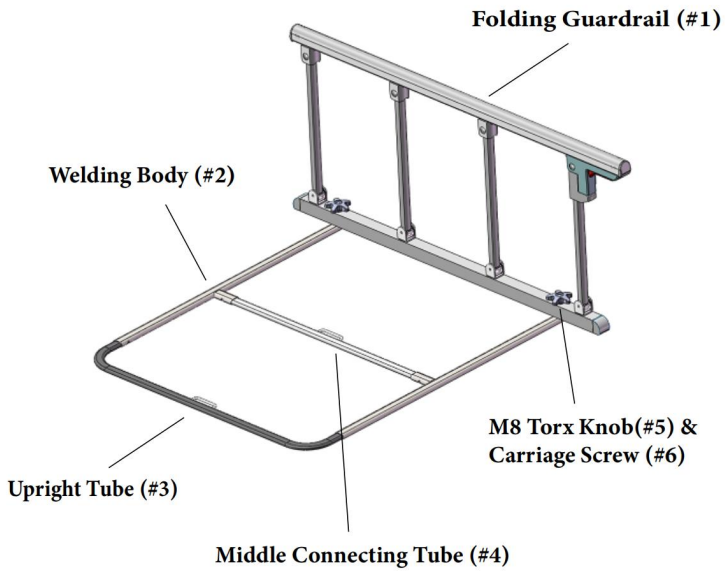

Parts List


Part Number	Diagram	Name	Quantity(Pcs)
#1		Folding Guardrail	1
#2		Welding Body	2
#3		Upright Tube	1
#4		Middle Connecting Tube	1
#5		M8 Torx Knob	2

#6		Carriage Screw	2
#7		Safety frame strap	1

Installation


STEP 1

Please assemble according to the sticker A,B,C, and D. Assemble A-A, B-B with the Welded Body (#2) connecting the two side frames against the corresponding marble hole of the Middle Connecting Tube (#4) as indicated by the arrow.

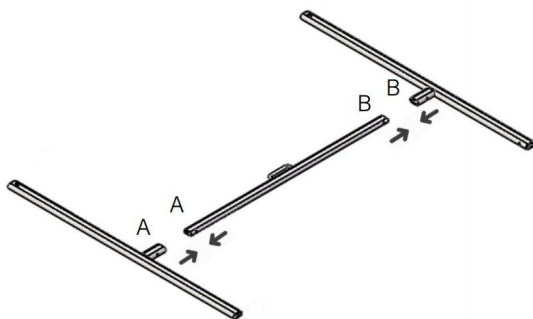


STEP 2

Assemble C-C, D-D, with the Upright Tube (#3) facing the corresponding marble holes of the side frame Welding Body (#2) as shown by the arrow.

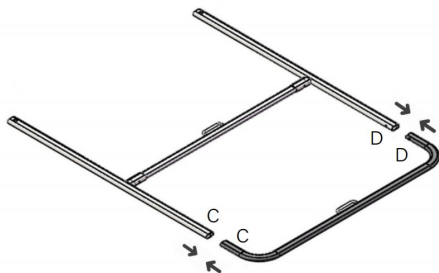


STEP 3

Insert the Carriage Screw (#6) into the square hole of the side frame welding body and pass through the hole of the folding guardrail, and lock it with M8Torx Knob(#5).

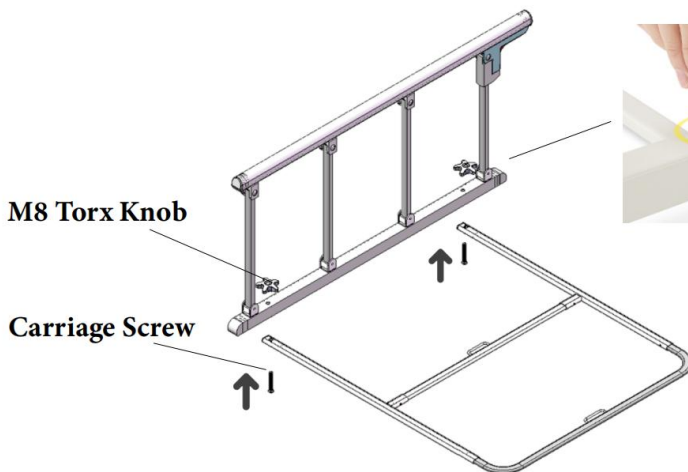

STEP 4


Use the provided safety frame strap (#7) to secure the bed rails to the bed frame on the opposite side of the bed through the U-shaped slots (#4, #3).

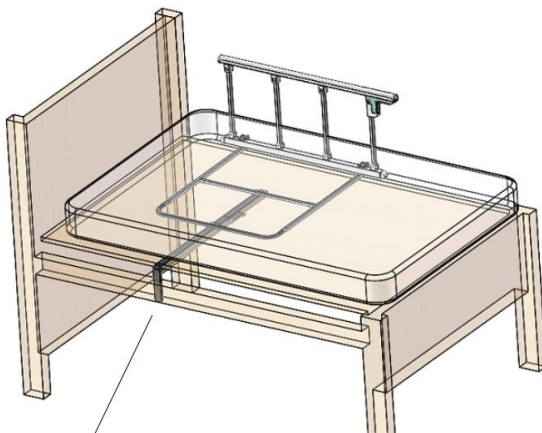

Safety Frame Strap (#7)

Special Note

1. If you need to lift up the guardrail when the Bed Safety Rail is in a folded state, hold the red switch and lift it up until you hear a "click", that is, the switch marble snaps into place, then release the switch.
2. If it needs to be folded when the Bed Safety Rail is in an upright state, hold the red switch and press down to fold it.

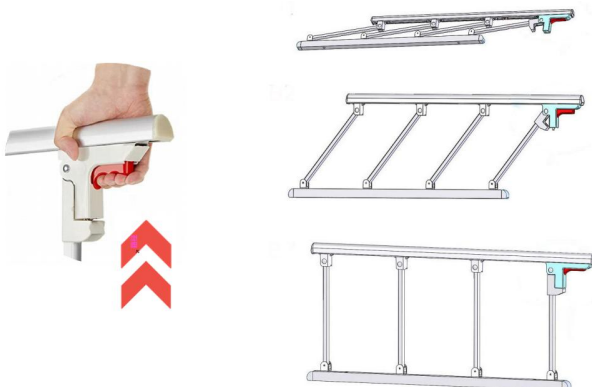

Safety Instructions

1. To confirm the Bed Safety Rail is installed correctly, please refer to the diagram in the manual for comparison.
2. Verify that all accessories are fully assembled and check for redundant accessories.
3. Check whether there is any looseness in the installation, and install it if it is loose.
4. Check the appearance of all components for skew, and make appropriate adjustments.

Maintenance

To help you maintain your Bed Safety Rail in optimal condition please refer to our suggestions below.

1. Do not scratch the surface of the product with hard objects to prevent damage to the material.
2. Do not contact with strong acid and strong alkali to prevent corrosion.
3. For daily cleaning and maintenance, use a neutral synthetic detergent, wipe with a damp cloth, and then wipe with a dry cloth.

WARNING

Serious injury or death can result if bed rails are not properly installed. Entrapment has resulted in injuries and death, Entrapment occurs when an individual becomes wedged between a bed rail and any other object. It is possible for an individual's head body, arms, or legs to be trapped between the bars of a bed rail, between the bed rail and the head or foot board of a bed, or between the bed rail and a mattress.

While there are no specific guidelines for bed rails is committed to providing safe products and informing the user of ways to prevent accidents.

SAFETY PRECAUTIONS

1. Regularly check the bed rail to identify areas of possible entrapment.
2. Position the bed rails away from head or foot boards of a bed.
3. Make sure that there is no gap between the bed rail and the side of the mattress. A gap could allow the user to become wedged between the bed rail and the mattress.
4. Adjust the height of the bed rail handle to avoid small gaps between the bed rail handle and the top of the mattress.
5. The product instructions must be carefully followed to ensure that the product is properly assembled and positioned. Failure to follow the instructions could result in serious injury to the user(s) of the product.
6. Children should not be allowed to hang on or play with the product.

Manufacturer: FOSHAN HUAFUKANG MEDICAL TECHNOLOGY CO. LTD.

Address: Youhe Industry Area, Luoxing, Danzao Town, Nanhai District, Foshan City, Guangdong, 528216, CHINA.

E-mail: support@vevor.com

