



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900977


Technical Support and E-Warranty Certificate
www.vevor.com/support



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900977

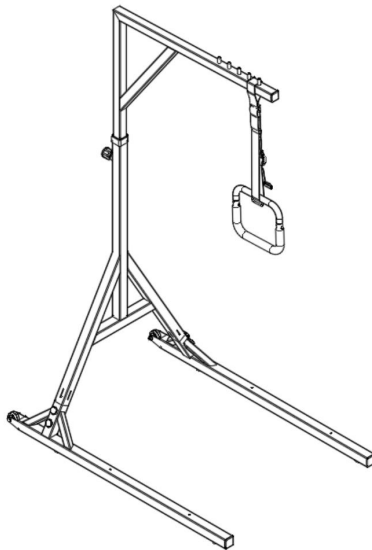

Note: The product picture is for reference, the actual details shall prevail

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Patient Transfer Chair

Please read all of the instructions before using it. The information will help you achieve the best possible results.

Operation safety

Please put this manual in a place easy to check and keep it properly. This instruction manual clearly indicates how to use the equipment safely and correctly, so as to prevent any harm or property loss caused fro you and others.



Danger



Warning

Weight Capacity: 350 lbs. Never use the stand assist for patients exceeding this weight.

- Only operate where permitted according to all applicable local laws and ordinances.
- Only qualified caregivers trained in the proper use of stent AIDS should use them.
- The Bed Trapeze Bar should only be used to assist the patient to sit on the bed. Stent assistance should not be used to transfer patients from one location to another.
- Before using the equipment, check all parts for shipping damage. If any part is damaged, do not use it, and contact VEVOR for further instructions.
- Before operating the stand assist, ensure that all components are properly and securely assembled and that all fasteners are secure.
- The rear caster must be locked before lowering the patient into the stand assist for transport.
- Avoid over tightening mounting hardware as this may cause damage to mounting brackets
- The stand assist may tip over if used incorrectly. Read and adhere to the operating instructions before transferring anyone.
- Exercise caution if leaving the stand assist under direct sunlight for an extended period as its surface may become hot.





Do not use for the purpose of transportation over long distances

Do no use for the purpose of seating over long extended periods Do not use this product if it is damaged, and do not use attachments or substitute parts Regular

maintenance of stand assist and accessories is necessary to ensure proper operation.


If you give this product to someone else, must include this owner's manual.

Part List

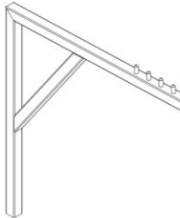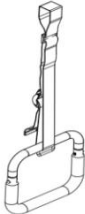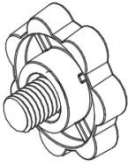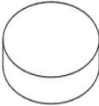01.  1PCS	02.  1PCS	03.  1PCS	04. M8*8mm  5PCS
05.  1PCS	06.  5PCS	07.  1PCS	08.  1PCS
09.  1PCS	10.  1PCS	11.  5PCS	

NOTE: Some fasteners will have an emergency spare part .


Explosion diagram

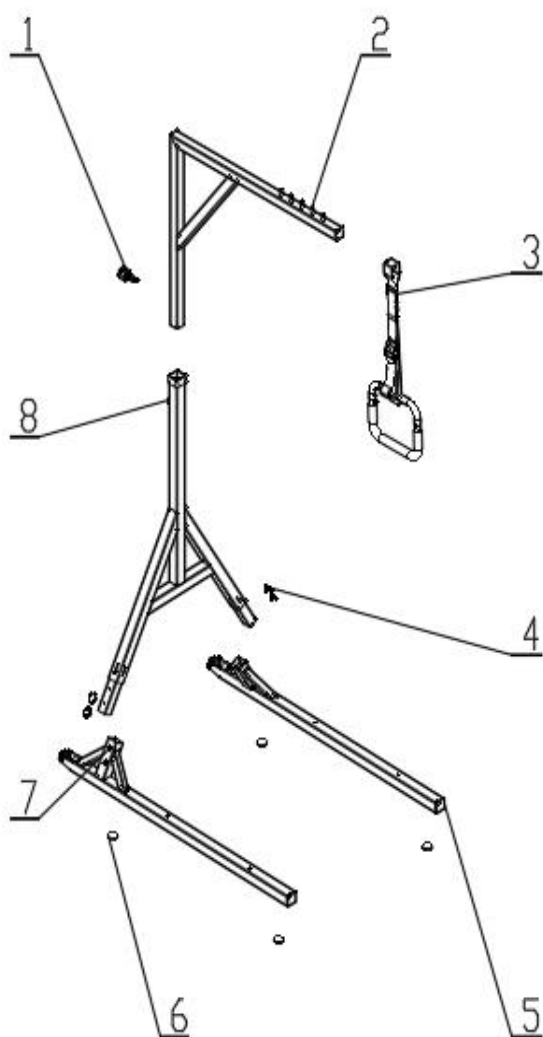

Assembly Steps


Step1

- ① Tear the $\phi 32 \times 8\text{mm}$ (06) backing glue of four rubber pads and paste them on the right component of the support foot (05) and the left component of the support foot (07) respectively at the appropriate positions at both ends (it is recommended to paste them in the center 60mm and 860mm away from the non-wheel end).
- (2) Place the right component (05) of the support foot parallel to the right component (07) of the support foot at the appropriate position (0.7m recommended); Then insert one foot of the connecting tube assembly (08) into the left component of the supporting foot (07) in the direction shown, then tilt the connecting tube assembly (08), move the right component of the supporting foot (05) to the other foot of the connecting tube assembly (08), and insert; Tighten with a plum knob (M8X8mm) (04).

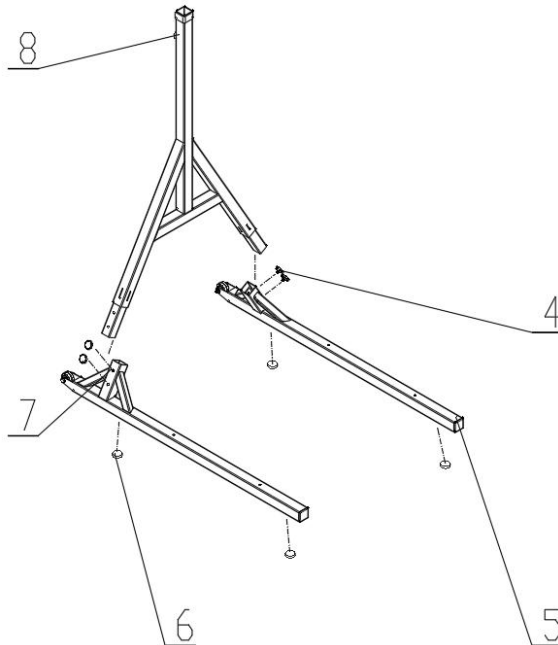

Step2

- ① Align the upper tube assembly (02) with the corresponding hole position of the connecting tube assembly (08), and then tighten it with the strong pull pin knob (01);
- ② Hang the lifting ring assembly (03) between the columns of the upper pipe assembly (02).

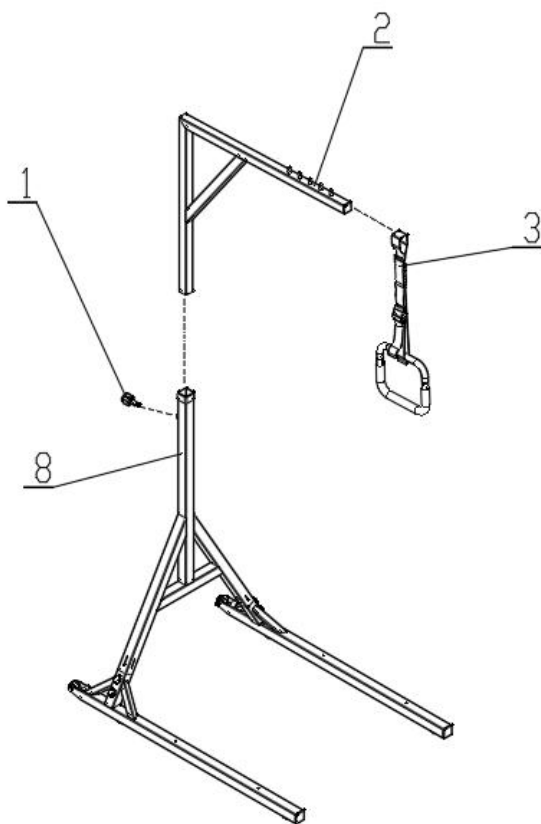

Tips:

After the assembly is complete, place the right component of the support foot (05) and the left component of the support foot (07) under the bed, and move the hanger into the bed as a whole. The lifting ring can be adjusted to a comfortable position by adjusting the lifting ring assembly (03) between the columns of the upper pipe assembly (02).

Loosen the power pull pin knob (01) (without screwing out), then pull the power pull pin knob (01), and raise or lower (between 1 and 3 limit lines) the upper tube assembly (02) about 2cm, release the hand pulling the power pull pin knob (01), then continue to move in the previous lifting direction until stuck. The overall height of the hanger can be adjusted through this operation

OPERATING INSTRUCTIONS

Operation Precautions

- Do not exceed the recommended weight limit of 350 lbs
- Before use, make sure all assemblies are securely tightened and thoroughly read the operating instructions

CARE AND MAINTENANCE

Cleaning:

- Wipe down the frame with a soft dry cloth at least once a week. If the stand assist comes into contact with moisture, dry it immediately.

Notice:

- Do not use solvents, abrasive waxes, caustic chemicals or spray silicone to clean the equipment.
- Do not use abrasive cleansers to clean the equipment, they could scratch the finish.
- Do not use steam or high-pressure cleaners to clean the equipment.

Maintenance:

To ensure safety and proper use, perform the following steps monthly: Check all fasteners for tightness and wear. Replace any worn or loose fasteners before use. Check the seat assemblies for damage or wear, and ensure bolts are tight. Replace any worn or damaged components. Check the casters are functioning correctly and securely attached. Replace any worn or damaged casters

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

E-CrossStu GmbH
Mainzer Landstr.69,
60329 Frankfurt am Main.



Technical Support and E-Warranty Certificate
www.vevor.com/support