



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900976

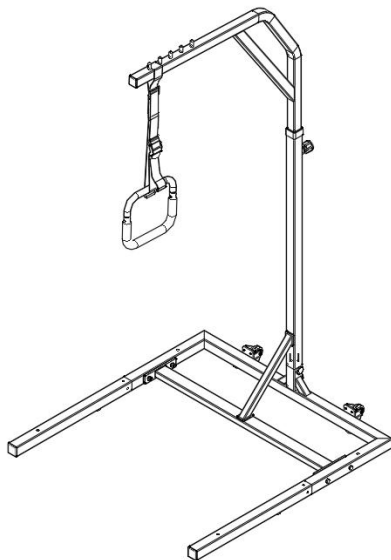
Technical Support and E-Warranty Certificate
www.vevor.com/support



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900976



Note:The product picture is for reference, the actual details shall prevail

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Patient Transfer Chair

Please read all of the instructions before using it. The information will help you achieve the best possible results.

Operation safety

Please put this manual in a place easy to check and keep it properly. This instruction manual clearly indicates how to use the equipment safely and correctly, so as to prevent any harm or property loss caused for you and others.



Danger



Warning

Weight Capacity: 300 lbs. Never use the stand assist for patients exceeding this weight.

- Only operate where permitted according to all applicable local laws and ordinances.
- Only qualified caregivers trained in the proper use of stand assist should use them.
- The Bed Trapeze Bar should only be used to assist the patient to sit on the bed. Stand assist should not be used to transfer patients from one location to another.
- Before using the equipment, check all parts for shipping damage. If any part is damaged, do not use it, and contact VEVOR for further instructions.
- Before operating the stand assist, ensure that all components are properly and securely assembled and that all fasteners are secure.
- The rear caster must be locked before lowering the patient into the stand assist for transport.
- Avoid over tightening mounting hardware as this may cause damage to mounting brackets
- The stand assist may tip over if used incorrectly. Read and adhere to the operating instructions before transferring anyone.
- Exercise caution if leaving the stand assist under direct sunlight for an extended period as its surface may become hot.

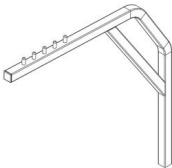
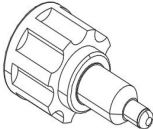
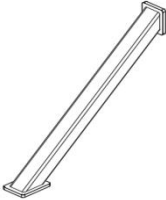
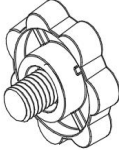
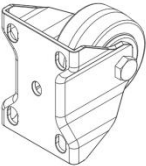
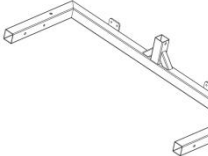
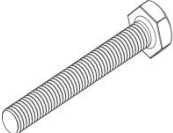
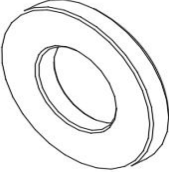
Do not use for the purpose of transportation over long distances





Do not use for the purpose of seating over long extended periods Do not use this product if it is damaged, and do not use attachments or substitute parts Regular

maintenance of stand assist and accessories is necessary to ensure proper operation.

If you give this product to someone else, must include this owner's manual.


Part List

01.  1PCS	02.  1PCS	03.  1PCS	04.  1PCS
05. M6*12mm  9PCS	06. M8*48mm  1PCS	07. M8*8mm  3PCS	08.  2PCS
09.  1PCS	10. M8*60mm  5PCS	11.  5PCS	12.  5PCS
13.	14.	15.	16.

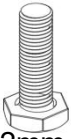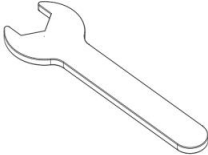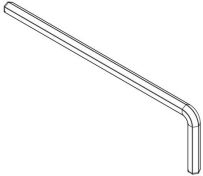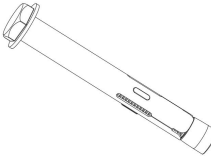 M8*32mm 1PCS	 5PCS	 2PCS	 1PCS
17.  1PCS	18.  2PCS	19.  1PCS	20.  1PCS
21.  5PCS			

NOTE: Some fasteners will have an emergency spare part

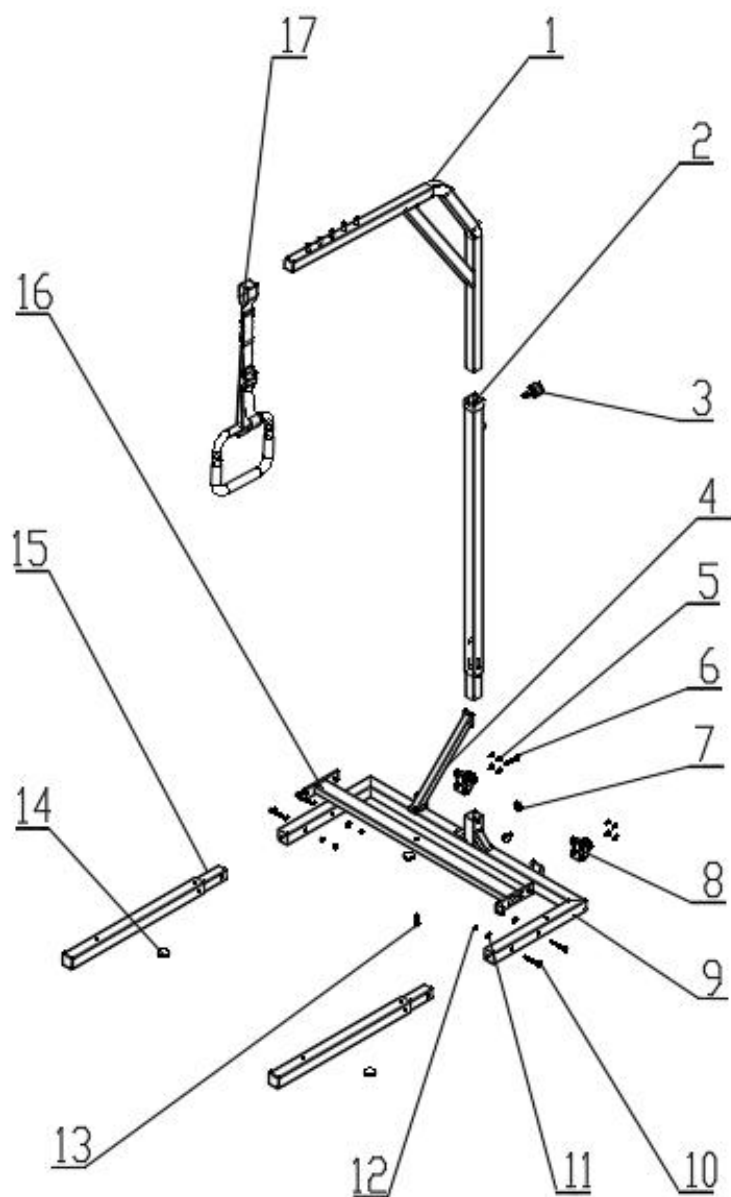
Explosion diagram



Assembly Steps


Step1

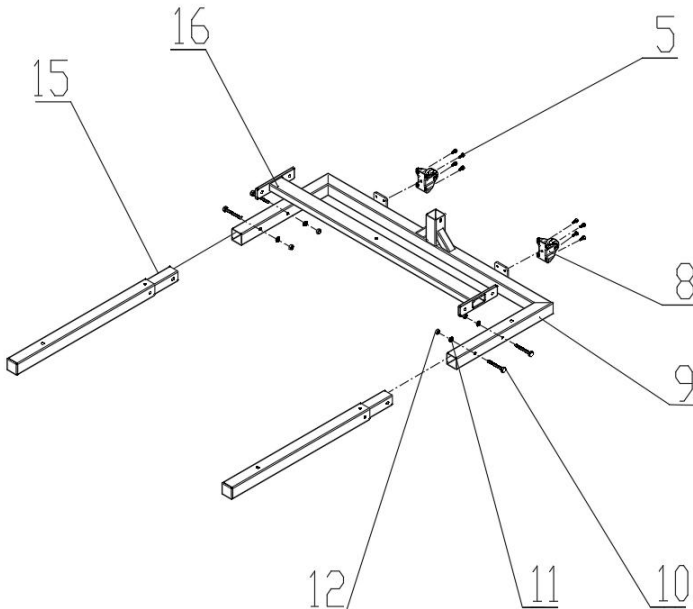- ① Align the two 1.5-inch directional wheel assembly (08) with the hole position of the wheel mounting plate of the U-shaped base assembly (09) and fix them with four cylindrical head hexagon screws M6X12mm (06).
- ② Insert the two base extension tube assemblies (15) into the two foot tubes of the U-shaped base assembly (09) respectively at the hole positions; Then place the base connecting tube assembly (16) in the middle of the U-shaped base assembly (09), align the hole, and pass the U-shaped base assembly (09), base connecting tube assembly (16), flat washer (diameter 16X 8.2X2) (11), and ordinary hexagon nut (M8) (12) through the outer hexagon screw (M8X60mm) (10). Pretighten and repeat four times.




Step2

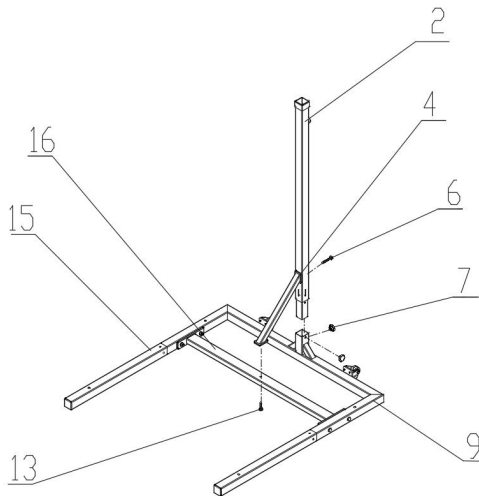- ① Align the standpipe assembly (02) with the standpipe hole (as shown in the following figure) of the U-shaped base assembly (09) and insert it, then screw in the two plum knobs (M8X8mm) (07) (do not tighten).
- ② Align the screw holes of the base inclined support assembly (04) with the corresponding holes of the standpipe assembly (02) and the base connecting pipe assembly (16), and insert the external hexagon screw (M8X32mm) (13) through the base connecting pipe assembly (16) and the base inclined support assembly (04) and partially tighten them. Use the outer hexagon screw (M8X48mm) (06) to pass through the standpipe assembly (02) and the base diagonal support assembly (04) and pretighten. Then tighten the two club knobs (M8X8mm) (07).
- ③ Tighten with a wrench to ensure that all nuts, bolts and components are secured in place before use.




Step3

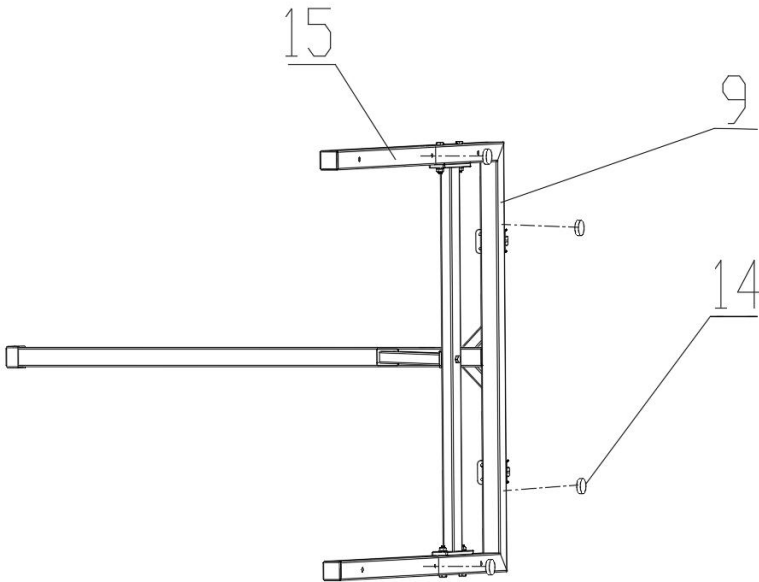- ① Place the assembled U-shaped base assembly (09) on the side, Tear off four rubber pads with a diameter of 32X8mm (14) and attach them to appropriate positions on both sides of the base extension tube assembly (15) and U-shaped base assembly (09) respectively (it is recommended to paste them in the center 40cm from the outermost end of the base extension tube assembly (15) and 18cm from the side of the U-shaped base assembly (09)).
- ②After attaching four rubber pads with a diameter of 32X8mm (14), straighten the U-shaped base assembly (09)



Step4

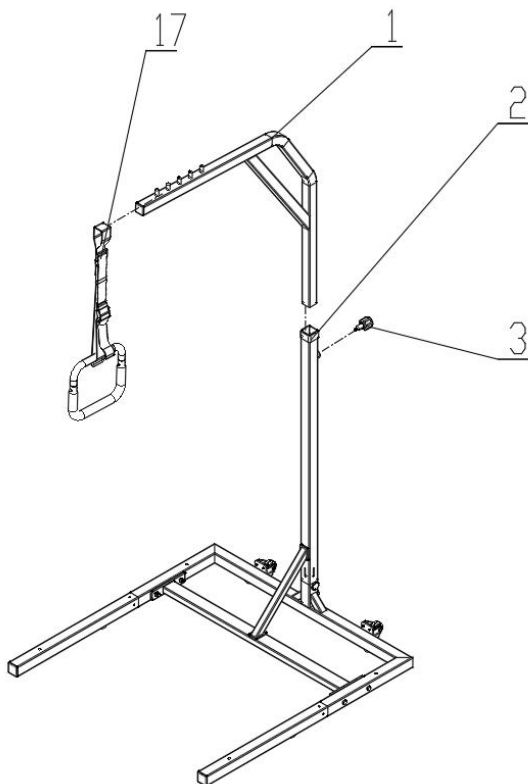- ① Align the upper pipe assembly (01) with the corresponding hole position of the standpipe assembly (02), and then tighten it with the strong pull pin knob (03).
- ② Hang the lifting ring assembly (17) between the columns of the upper pipe assembly (01)



Tips:

1.After assembly, place the two base extension tube assemblies (15) under the bed and move the hanger into the bed as a whole. The lifting ring can be adjusted to a comfortable position by adjusting the position of the lifting ring assembly (17) between the columns of the upper pipe assembly (01).

2.Loosen the power pull pin knob (03) (without screwing out), then pull the power pull pin knob (03), and raise or lower (between 1 and 3 limit lines) the upper tube assembly (01) about 2cm, release the hand pulling the power pull pin knob (03), then continue to move in the previous lifting direction until stuck. The overall height of the hanger can be adjusted through this operation

OPERATING INSTRUCTIONS

Operation Precautions

- Do not exceed the recommended weight limit of 300 lbs
- Before use, make sure all assemblies are securely tightened and thoroughly read the operating instructions

CARE AND MAINTENANCE

Cleaning:

- Wipe down the frame with a soft dry cloth at least once a week. If the stand assist comes into contact with moisture, dry it immediately.

Notice:

- Do not use solvents, abrasive waxes, caustic chemicals or spray silicone to clean the equipment.
- Do not use abrasive cleansers to clean the equipment, they could scratch the finish.
- Do not use steam or high-pressure cleaners to clean the equipment.

Maintenance:

To ensure safety and proper use, perform the following steps monthly: Check all fasteners for tightness and wear. Replace any worn or loose fasteners before use. Check the seat assemblies for damage or wear, and ensure bolts are tight. Replace any worn or damaged components. Check the casters are functioning correctly and securely attached. Replace any worn or damaged casters

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

E-CrossStu GmbH
Mainzer Landstr.69,
60329 Frankfurt am Main.



Technical Support and E-Warranty Certificate
www.vevor.com/support