



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900978

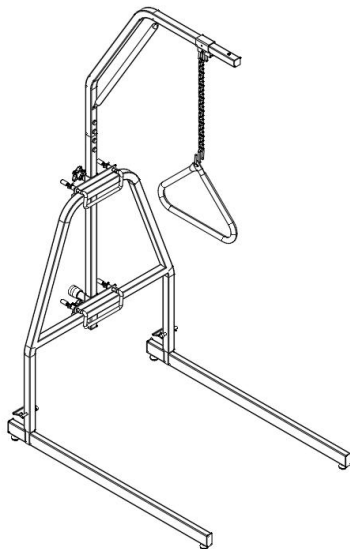
Technical Support and E-Warranty Certificate
www.vevor.com/support



Affordable. Reliable. Home Improvement.

Bed Trapeze Bar

MODEL: FBL900978



Note:The product picture is for reference, the actual details shall prevail

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Patient Transfer Chair

Please read all of the instructions before using it. The information will help you achieve the best possible results.

Operation safety

Please put this manual in a place easy to check and keep it properly. This instruction manual clearly indicates how to use the equipment safely and correctly, so as to prevent any harm or property loss caused for you and others.



Danger



Warning

Weight Capacity: 250 lbs. Never use the stand assist for patients exceeding this weight.

- Only operate where permitted according to all applicable local laws and ordinances.
- Only qualified caregivers trained in the proper use of stand assist should use them.
- The Bed Trapeze Bar should only be used to assist the patient to sit on the bed. Stand assist should not be used to transfer patients from one location to another.
- Before using the equipment, check all parts for shipping damage. If any part is damaged, do not use it, and contact VEVOR for further instructions.
- Before operating the stand assist, ensure that all components are properly and securely assembled and that all fasteners are secure.
- The rear caster must be locked before lowering the patient into the stand assist for transport.
- Avoid over tightening mounting hardware as this may cause damage to mounting brackets
- The stand assist may tip over if used incorrectly. Read and adhere to the operating instructions before transferring anyone.
- Exercise caution if leaving the stand assist under direct sunlight for an extended period as its surface may become hot.

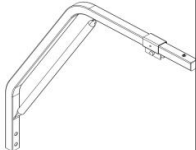
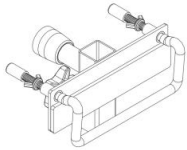
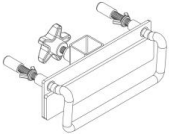
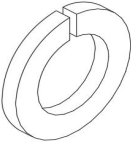
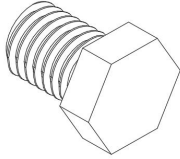
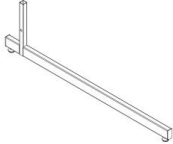
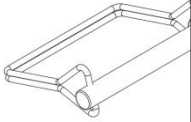
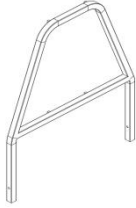
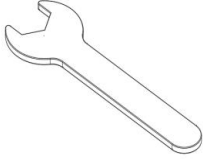
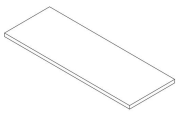
Do not use for the purpose of transportation over long distances

Do not use for the purpose of seating over long extended periods Do not use this product if it is damaged, and do not use attachments or substitute parts Regular

maintenance of stand assist and accessories is necessary to ensure proper operation.


If you give this product to someone else, must include this owner's manual.

Part List

01.  1PCS	02.  1PCS	03.  1PCS	04.  1PCS
05.  1PCS	06.  1PCS	07.  4PCS	08.  4PCS
09.  2PCS	10.  2PCS	11.  1PCS	12.  1PCS
13 Eva Pad  1PCS			

NOTE: Some fasteners will have an emergency spare part

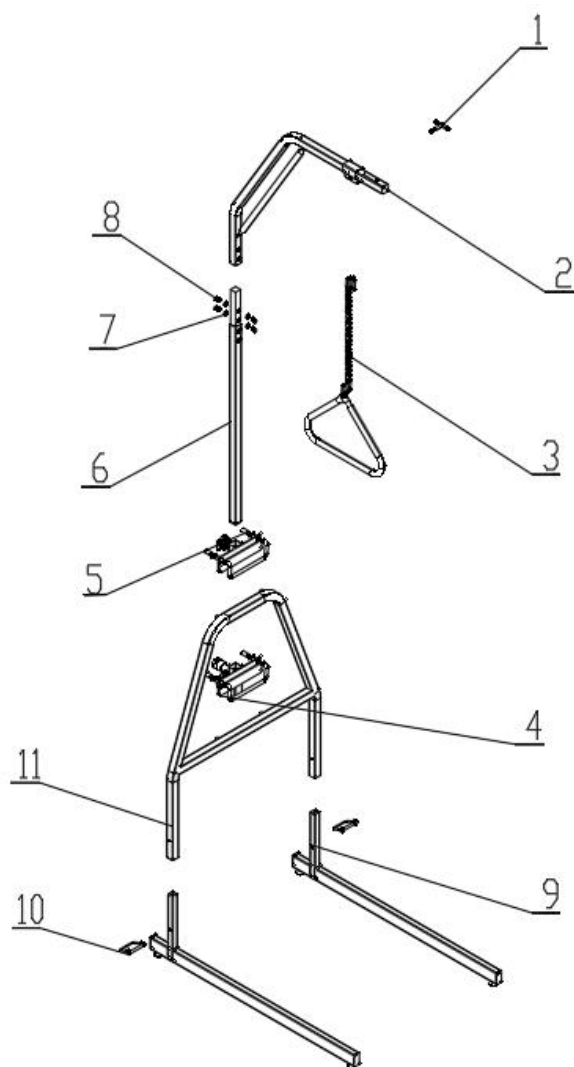
Explosion diagram



Assembly Steps


Step1

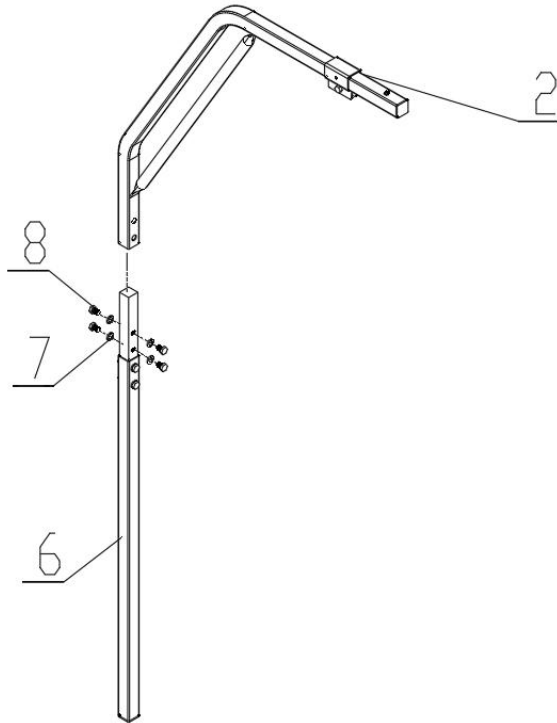Twist the four outer hex screws M10x16 (08) from the hanger standpipe assembly (06) and remove the four spring washers (07) with the outer hex wrench (with M10 screws) (12); Then insert the upper tube assembly (02) into the hanger standpipe assembly (06). After positioning the holes, screw the four outer hexagonal screws M10x16 (08) into the hanger standpipe assembly (06) through the spring washer (07); Finally, use an external hex wrench (with M10 screws) (12) to tighten.



Step2

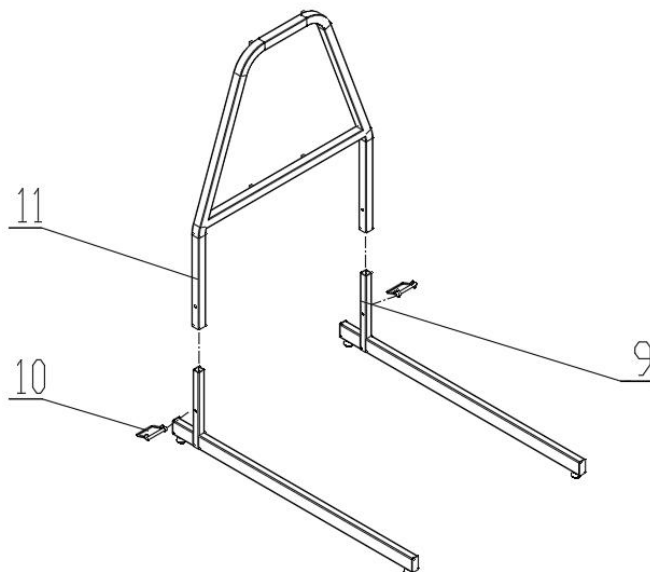Place the two tripod assemblies (09) in parallel at a distance of 68.5cm, and then align the two feet of the trapezoidal fixture assembly (11) with the standpipe of the two tripod assemblies (09) as shown in the figure; Finally, after the hole position is aligned, lock it with the card cable latch (10).




Step3

① Place the clamping assembly A (05) and the clamping assembly B (04) on the transverse tube of the trapezoidal fixing assembly (11) as shown in the figure;


② Insert the hanger standpipe assembly (06) into the short pipe of clamping assembly A (05) and clamping assembly B (04) successively as shown in the following figure, and then tighten the knobs of clamping assembly A (05) and clamping assembly B (04) to fix them on the hanger standpipe assembly (06);

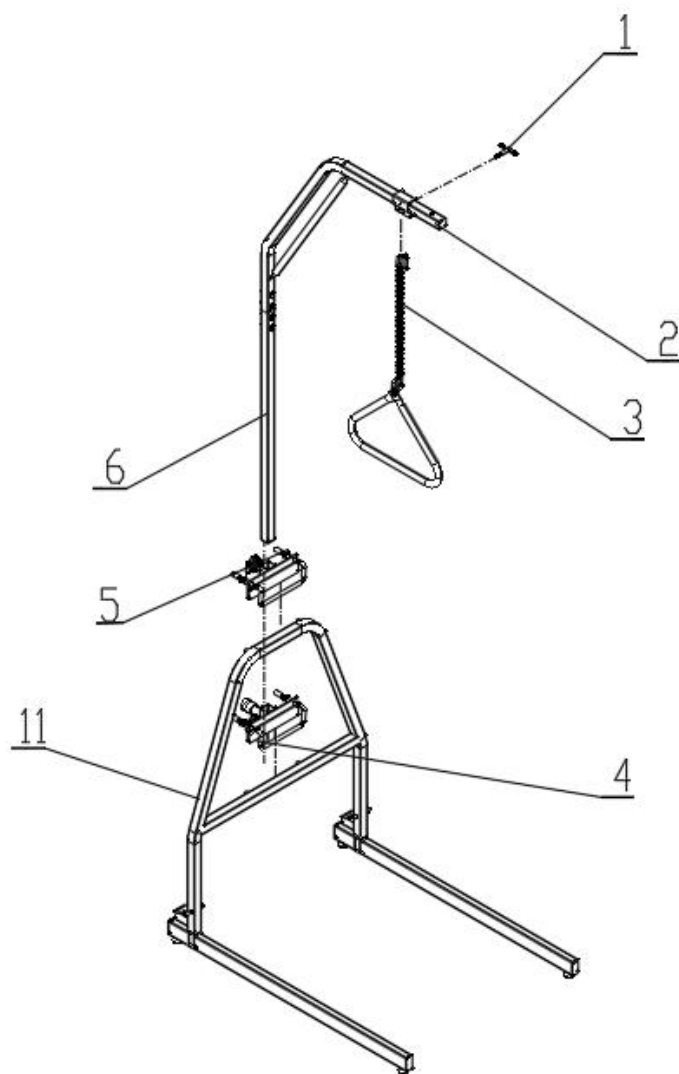③ Place the lifting ring assembly (03) into the clamp of the upper tube assembly (02) as shown in the figure below, and then tighten the locking assembly (01) through the clamp of the upper tube assembly (02) and the lifting ring assembly (03) in turn.



Step4

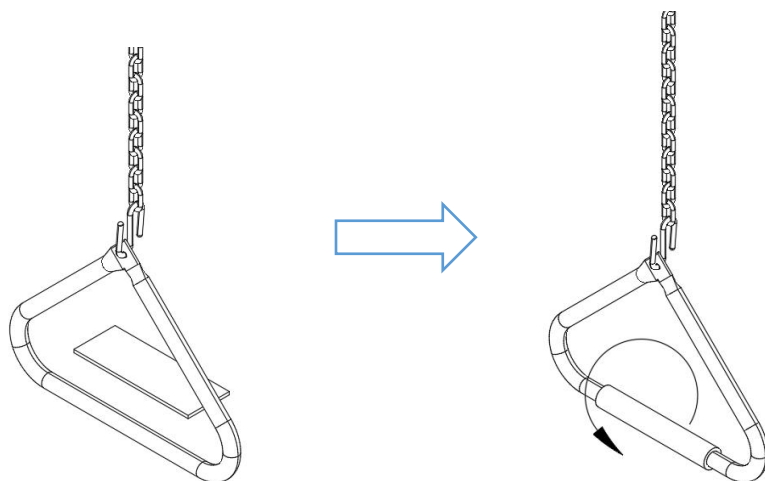After the assembly is completed, the EVA pad (13) can be ripped off the sticker and pasted on the grip of the ring as needed.



Tips:

Place the tripod assembly (09) under the bed and move the whole hanger into the bed. The knobs of the clamping assembly A (05) and the clamping assembly B (04) can be loosened to adjust the height of the hanger by moving the standpipe assembly (06) up and down (note: the clamping assembly A (05) needs to move between the (MIN) and (MAX) limit lines).

The clamp of the upper tube assembly (02) can be loosened by loosening the locking assembly (01) so that the clamp of the upper tube assembly (02) is no longer clamping the tube of the upper tube assembly (02), and then the clamp of the upper tube assembly (02) can be moved back and forth,

and the locking assembly (01) can be tightened again after being in a suitable position so that the clamp of the upper tube assembly (02) is clamping the upper tube assembly (02)

OPERATING INSTRUCTIONS

Operation Precautions

- Do not exceed the recommended weight limit of 250 lbs
- Before use, make sure all assemblies are securely tightened and thoroughly read the operating instructions

CARE AND MAINTENANCE

Cleaning:

- Wipe down the frame with a soft dry cloth at least once a week. If the stand assist comes into contact with moisture, dry it immediately.

Notice:

- Do not use solvents, abrasive waxes, caustic chemicals or spray silicone to clean the equipment.
- Do not use abrasive cleansers to clean the equipment, they could scratch the finish.
- Do not use steam or high-pressure cleaners to clean the equipment.

Maintenance:

To ensure safety and proper use, perform the following steps monthly: Check all fasteners for tightness and wear. Replace any worn or loose fasteners before use. Check the seat assemblies for damage or wear, and ensure bolts are tight. Replace any worn or damaged components. Check the casters are functioning correctly and securely attached. Replace any worn or damaged casters

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 2F-2, 3-6F & 7F-1, No. 63,Tongxing East Road, Yumin Community,
Xiaolan, Zhongshan, Guangdong, 528414, CHINA

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122
Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,
Rancho Cucamonga, CA 91730



YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX



E-CrossStu GmbH

Mainzer Landstr.69,
60329 Frankfurt am Main.



Technical Support and E-Warranty Certificate
www.vevor.com/support